
# Esteem™ Flex Convex

Instructions for use / Bruksanvisning / Gebrauchsstructurce / Návod k použití / Návod na použitie / Instruccia stosowania / Istruzioni per l'uso / ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ / Brugsanvisning / Käyttöohjeet / Användnings / Kullannin Talimat / Naučimoje instrukcija / Uputstvo za upotrebu / Upute za uporabu / Lietošanas pamācība / Navodila za uporabo / Kasutusjuhend / Használati utasítás / Gebrauchsanleitung / Notice d'utilisation / إرشادات الاستخدام / Instrucciones de uso / Instrucțiuni de utilizare / Instruções de utilização

## III ConvaTec

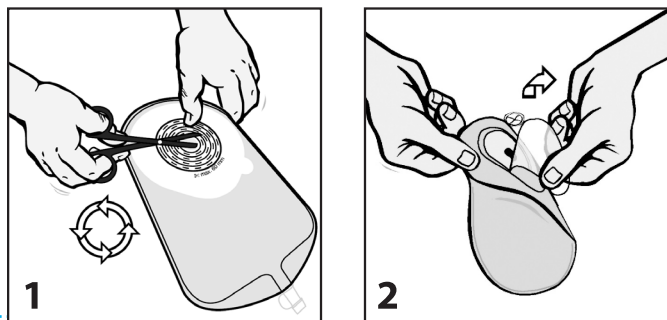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**8. Advies voor use of a drainable pouch with Safe Seal Closure**  
For those with an ileostomy or colostomy there is a type of pouch that can be drained, the tall closure cleaned and sealed again.

**Important:** A healthcare professional should assess the need for skin barriers with convexity prior to use.

**DIRECTIONS FOR USE**

**1. Preparation**  
Clean the skin around your stoma with water and a soft cloth. Pat the area dry. Do not use oil-based products as they may interfere with the adhesion of the skin barrier.

Please make sure you know the diameter of your stoma. Esteem™ Flex Convex is available in pre-cut and cut-to-fit options.

For cut-to-fit pouches you can use the cutting guide on the water holder.

Partially inflate the pouch by blowing to separate the inner layers of the pouch. Warm the skin barrier with the palm of your hand for a few seconds to aid adhesion. Remove the plastic covering from the skin barrier (figure 2). Try to avoid touching the adhesive area too much with your hands.

Hold the skin barrier below your stoma. Beginning from the bottom carefully adhere the barrier to the skin around your stoma (figure 3).

Smooth over the back of the adhesive area with your fingers and press carefully but firmly around your stoma (figure 4).

**3. Advice for use of a drainable pouch with Safe Seal Closure**  
For those with an ileostomy or colostomy there is a type of pouch that can be drained, the tall closure cleaned and sealed again.

Please note the following advice for using a drainable pouch with Safe Seal closure:

Make sure the bottom of the pouch is securely fastened. Fold up the neck of the pouch and continue to use the palm of your hand for a few seconds to aid adhesion. In contact with each other. Press firmly together to gain a secure seal (figure 5).

Apply the pouch as instructed. Pull the tab to open the closure and empty the pouch.

**4. Advice for using a urostomy pouch**  
Make sure the bottom of the pouch is securely fastened. Apply the pouch as instructed. Use the supplied adaptor to connect the urostomy pouch to a night drainage pouch or leg bag if necessary (figure 5).

**5. Information about pouches with filters**  
There is a built in filter for closed and drainable pouches through which gases are vented and neutralized. The filter sticker provides serious as effective protection against moisture (e.g. when bathing, showering) and ensures thereby the functional performance of the filter.

Cover the filter with the sticker when bathing, swimming, etc (figure 7). Remove after water activity.

**6. Pouch removal and disposal**  
To remove an old pouch or drainage bag, gently peel downwards while holding the skin above the water with your free hand (figure 8). This helps avoid skin irritation and makes the removal easier.

Empty the contents of the pouch into the toilet.  
**Never flush the pouch down the toilet.**  
After use, this product may be a potential biohazard. Handle and dispose in accordance with accepted medical practice and applicable local laws and regulations.

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# Effective

ARTWORK SPECIFICATION	
Artcode	SI2222VC
Artwork Description	INS EST+ FLEX CVX V1 PCUT DRN EUR
ICC*	421612, 421613, 421614, 421615, 421616, 421617, 421618, 421619, 421620, 421621, 421622, 421623, 421624, 421625, 421626, 421627, 421628, 421629, 421612, 422158, 422159
SAP*	1714914, 1714916, 1714918, 1714920, 1714922, 1714924, 1714926, 1714928, 1714930, 1714932, 1714934, 1714936, 1714938, 1714940, 1714942, 1714908, 1714910, 1714912, 1719732, 1721237, 1721239
KOM/REF*	N/A
IS Code*	N/A
Component Type	Insert
LR#	LR-004578
Existing Art Code being Revised	SI2222VB
<b>REQUIREMENT</b>	<b>SPECIFICATION</b>
Bucket	EUR
Local Code(s)	N/A
Size Code / Dieline	N/A
Dimensions	Flat: 24 x 28 (609.6mm x 711.2mm) Folded: 4 x 5.6 (101.6mm x 142.2mm)
Color(s) / Colour(s)	Black
Specification Reference(s)	TF21-037
Bar Code: UPC (A) or GTIN**	N/A
Bar Code: PZN – Code 39**	N/A
Bar Code: JAN-13**	N/A
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.	
<b>HISTORY</b>	
<p>14-JULY-2016 / SI2222: New IFU for Esteem+ FlexConvex V1 PCUT DRN (EUR)</p> <p>29-SEPT-2016 / SI2222: IFU fold dimensions changed to 4 x 5.6 (101.6mm x 142.2mm). No change to text. Documentation change only.</p> <p>24-APR-2017 / SI2222: Add SAP Code 1719732. No change to art. Documentation updates only.</p> <p>9-JUNE-2021 / SI2222VA: All symbols updated in accordance with ISO 15223-1:2016 and MDR compliance. Added language codes. Added MD symbol and the corresponding definition in all languages required. Added statements in the IFU Intended use and indication for use and it's translation acc. to SM-076885. Updated IFU statements regarding the device single use, during and after use according to SM-074623. Art version changed to VA. Copyright and revision date updated. Updated Deeside address and manufacturing symbol. Added importers symbol and explanation. EC REP symbol with address added with importer symbol. This version was never implemented.</p> <p>06-OCT-2021/SI2222VB: Indication for use updated to indicate " For use with a colostomy, ileostomy or urostomy" and it's corresponding translation.</p> <p>23-MAY-2022 / SI2222VC: Added CH REP and importer symbol and address.Turkey address updated. Art version and copyright date updated. Finished codes updated according to TF.</p>	