

Effective

ARTWORK SPECIFICATION	
Artcode	1716228V4
Artwork Description	INS NATURA LP CTF EUR
ICC*	421453, 421454, 421455, 421456, 421457, 421458, 421459, 421460, 421461, 421462, 421463, 421464, 423548, 423550, 423551
SAP*	1712824, 1712826, 1712828, 1712830, 1712832, 1712834, 1712836, 1712838, 1712840, 1712842, 1712844, 1712846, 1737975, 1735706, 1735708, 1735709, 1737999, 1738004, 1738007, 1738012, 1737972, 1737977, 1737991, 1737474, 1737988, 1737985, 1737980, 1746609
KOM/REF*	N/A
IS Code*	N/A
Component Type	Insert
LR#	LR-004919
Existing Art Code being Revised	1716228V4
REQUIREMENT	SPECIFICATION
Bucket	EUR
Local Code(s)	N/A
Size Code / Dieline	N/A
Dimensions	Flat: 24" x 28" / Folded: 4" x 4"
Color(s) / Colour(s)	Black
Specification Reference(s)	MT56-005
Bar Code: UPC (A) or GTIN**	N/A
Bar Code: PZN – Code 39**	N/A
Bar Code: JAN-13**	N/A
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.	
HISTORY	
<p>16-AUG-2016 / 1716228: New IFU artwork for Accordion Flange Cut To fit XL for HAI</p> <p>27-Sept-2016 / 1716228: SAP codes updated</p> <p>11-Oct-2016 / 1716228: Correct plant routing - Haina only /remove GBO.</p> <p>16-JUN-2017 / 1716228A1: Addition of English (en) and Arabic translations beneath the seven IFU symbols on the Accordion Flat Cut-to-Fit EUR insert. Specifically adding the following symbol translation for each for: 'Do not re-use', 'Caution', 'Consult instructions for use', 'Keep dry', 'Temperature limit', 'Catalogue number', and 'Batch code'. Removal of the phrase 'Produceret i USA' from the Danish (Dansk) translation. Update the copyright date in every translation from 2016 to 2017. Addition of Arabic as an additional language. Addition of Australia, Algeria (EMEA), and New Zealand telephone numbers. Adjusted font on insert to fit additional language. Updated artcode to A1 and issued date.</p> <p>26-JUN-2017 / 1716228A1: Added trademark symbol after Natura, Invisiclose, Durahesive, Stomahesive, and Accuseal where left out in insert. Inserted sponsor info for Australia.</p> <p>7-Mar-2018 / 1716228A1: Extension usage for new products. No artwork change, art spec update only. (approved but not implemented).</p> <p>16-Jun-2021 / 1716228V1: All symbols and their explanations updated acc. ISO 15223-1:2016 and MDR compliance, EC rep and importer address added. Added MD symbol and importer symbol with explanations, language codes. Update Convatec logo, CE mark, Deeside address, update intended use and indication for use acc. SM-074622, IFU sentences added acc. SM-074622. Update copyright and revision date to current year and version to V.</p> <p>07-Dec-2021 / 1716228V2: Added CH REP and importer symbol and address. Added symbol contains biological material of animal origin and symbol non-sterile and explanation and translation. Updated translation to "temperature limit" per SM-079379. Updated revision date, copyright and version.</p> <p>22-Sep-2022 / 1716228V3: Added French Canadian translation-text and symbols. Added address of Convatec Inc. and Canada phone number. Added english translation of symbols.</p> <p>12-May-2023 / 1716228V4: Added Trimann logo with explanations. Updated GBO address to full, including Canadian French translation of "manufactured for" and leaving only one address. Updated Australian address. Added legal manufacturer Canada address, remove all other CVT Canada Montreal address/number from rest of labeling. Added UKCA symbol. Added ICC code and SAP codes to ARTWORK SPECIFICATION.</p> <p>01-Jun-2023 / 1716228V4: Added sunlight symbol with explanation acc. SM-084218.</p> <p>22-Aug-2023 / 1716228V4: SAP codes updated.</p> <p>14-Mar-2024 / 1716228V4: Updated ICC and SAP codes in ARTWORK SPECIFICATION.</p>	