Natura™

Two-Piece Convex Ostomy System with Accordion Flange™

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION
The Natura™ "Durahesic" Skin Barrier with Accordion Flange™ system allows the flange to be placed between the skin barrier and the flange while coupling the pouch to the flange. It must be used with the same Natura™ "Natura™", "Sur-Fit™ Natura™" or "Natura™" adhesive.

INTENDED USE
Management of stoma output.

INDICATIONS FOR USE
Use with a colostomy, ileostomy or urostomy.

PRECAUTIONS AND OBSERVATIONS
Caution: Patients with a flush urostomy, when associated with impregnations and the absence of a urinary reservoir, should know that there have been reports of adhesive found in the urinary tract. Include appropriate clinical evaluation and evaluate the continued use of the adhesive product if a mass is detected.

CAUTION: This device is for single use only and should not be re-use. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the adhesive found in the urinary tract include appropriate clinical evaluation and evaluate the continued use of the adhesive product if a mass is detected.

CAUTION: A healthcare professional should assess the need for skin barriers with convulsory spasm. This device may cause skin irritation for users wearing diapers/wipes with incontinence protection. Consult your healthcare professional for more information.

CAUTION: Protect the stoma from trauma or injury.

DIRECTIONS FOR USE
1. Wash hands and the skin surrounding the stoma must be clean, dry and free from only substances before applying.

2. Skin Barrier Preparation
1) Select a water with a cutting range that is just larger than the widest diameter of the stoma. (Figure 1a)
2) Cut the skin barrier just above the flange. Do not cut beyond the outermost circle. (Figure 1a)
3) Run your finger along the inside of the flange to ensure any sharp edges. Do so until the release liner is still behind the adhesive mass.

3. Applying the Skin Barrier to the Skin Barrier
1) Remove the release liner from the skin barrier.
2) Center the opening of the skin barrier over the stoma.
3) Apply the skin barrier against the skin around the flange for 30 seconds.
4) Remove the remaining release liner from the fabric border.
5) Smooth in place and ensure the flange is secure.
6) Wear as long as comfortable and secure. Wear time may vary.

4. Applying the Pouch
1) Cut the pouch to allow air to enter. (Figure 3)
2) Hold the Accordion Flange™ system with three fingers on both hands.
3) Press the Accordion Flange™ system against the skin around the flange for 30 seconds.
4) Starting at the bottom, apply gentle pressure around the circumference of the flange until they "pop" in place and are fully attached.
5) Tap on the pouch to ensure a secure connection.

5. Closing the Pouch
1) Close the pouch beginning with the outlet end-strip until the interlocking closures appear. Press the interlocking closures together across the width of the flat collar and separate the two. Do not pull the pocket inside out to conceal the flat collar. (Pocket is available on Natura™ "pouches only")
2) Handle the Pouches with the Top Clip. (Figure 7)
3) Urostomy Pouches with Accu-Flush™ (Figure 10)
4) Urostomy Pouches with Soft Top™ (Figure 10)

6. Emptying the Pouch
1) Cover the flaps with InvisiClo™ Tail Closure (Figure 6)
2) Unhook the pouch closure and open the tail by pushing in both ends of the outlet end-strips with fingers. (Figure 6)
3) Drain the Pouches with the Top Clip. (Figure 12)
4) Urostomy Pouches with Soft Top™ (Figure 13)
5) Urostomy Pouches with Fold-over Top™ (Figure 14)
6) Empty the Pouches with the Top Clip. (Figure 12)

7. Connection to Night Drainage System
1) Connect to the Night Drainage System.
2) Follow the instructions for the Night Drainage System.
3) Soft Top™ (Figure 17)

8. Information about Pouches with Filters
1) Cover the filter with the InvisiClo™ Tail Closure when bathing, swimming, etc.
2) Remove after water activity.

9. Renewing the Pouch (Figure 18)
1) Remove the Skin Barrier (Figure 18)

10. Disposal
Please follow all local disposal regulations.

This device is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may not be optimal for intended use. After use, this product may be a potential biohazard. Handle and dispose of as intended with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and/or your national authority.

© 2023 Convatec Inc.
*Indicates trademarks of Convatec Inc.

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Effective

ARTWORK SPECIFICATION	
Artcode	1715214V3
Artwork Description	INS NATURA LP CVX CTF EUR
ICC*	421632, 421633, 421634, 421635, 421636, 421637, 421638, 421639, 421640, 421641, 421642, 421643
SAP*	1715163, 1715165, 1715167, 1715169, 1715171, 1715173, 1715175, 1715177, 1715179, 1715181, 1715183, 1715185, 1737995, 1738000, 1738003, 1738006, 1737759, 1737762
KOM/REF*	N/A
IS Code*	N/A
Component Type	Insert
LR#	LR-004693
Existing Art Code being Revised	1715214V2
REQUIREMENT	SPECIFICATION
Bucket	EUR
Local Code(s)	N/A
Size Code / Dieline	9989
Dimensions	Flat: 24" x 28" 609,6mm x 711,2mm / Folded: 4" x 4" 101,6mm x 101,6mm
Color(s) / Colour(s)	Black
Specification Reference(s)	MT56-005
Bar Code: UPC (A) or GTIN**	N/A
Bar Code: PZN – Code 39**	N/A
Bar Code: JAN-13**	N/A
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.	
HISTORY	
<p>29-JULY-2016 / 1713755: New Product Development -Natura Accordion Flange CVX CTF</p> <p>09-SEPT-2016 /1715214: Adjust and correctly align box around the LOT symbol. Art code will go to A version as art had already been sent to the print vendor.</p> <p>07-OCT-2016 / 1715214A1: Correct art code 1715214A to 1715214A1. No change to insert text - documentation change only.</p> <p>31-AUG-2021 / 1715214V1: Updated CE mark, updated symbols with explanations acc. ISO 15223-1, updated Convatec logo. Added language codes and updated language names, added specific text acc. SM-074622, updated indication for use and added intended use acc. SM-074622, added EC REP and importer symbol and address, added MD symbol and importer symbol with explanations, updated Australian and New Zealand contact information, updated version to V1, updated copyright date and revision date.</p> <p>18-Feb-2022 / 1715214V2: CH REP and importer symbol with address was added. Symbol contains biological material of animal origin with explanation and translations was added. Revision and copyright date were updated and version of artcode was increased to " V2".</p> <p>11-Mar-2022 / 1715214V2: Added UKCA mark.</p> <p>12-May-2023 / 1715214V3: Added Trimann logo with explanations for all products being sold in Europe. Updated Australian address. Added SAP codes to ARTWORK SPECIFICATION.</p> <p>01-Jun-2023 / 1715214V3: Added sunlight symbol with explanation acc. SM-084218.</p>	