EN

1a.

1b.

eakin perform™ solidifying agent

About this product
eakin perform™ is an absorbent powder contained in a water-soluble sachet which turns liquid stomal output to a thick gel. eakin perform™ may help reduce leaks and noise, providing increased security and discretion.

Intended users
For ostomates, healthcare professionals and caregivers.

Instructions for use
Insert one sachet of eakin perform™ into your pouch through the top opening (1.a). For drainable pouches, insert sachet(s) via the bottom opening each time you empty your pouch (1.b).

Warnings and cautions
Do not eat. Keep out of reach of children. Do not expose the sachet to open wounds. Do not rupture the sachet. If contact with the sachet contents occurs, flush with water. Seek medical advice if skin irritation occurs.

Do not re-use. Attempted re-use may expose the user to the sachet contents with a potential for skin irritation.

Other information
Store between 10° and 30° Celsius. Do not refrigerate. eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin® current recommendations. Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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© 2022 TG Eakin Ltd.