- en** Adapt Barrier Rings
- de** Adapt Hautschutzringe
- es** Anillos de barrera Adapt
- fr** Anneaux protecteurs Adapt
- fr-CA** Anneaux de champ protecteur Adapt
- it** Anelli barriera Adapt
- nl** Adapt-pastaringen

www.hollister.com/symbolglossary

912854

Hollister Incorporated  
2000 Hollister Drive  
Libertyville, IL 60048 USA

Hollister ULC  
EC REP  
Foxford Road, Ballina  
Co. Mayo, Ireland

©2020 Hollister Incorporated.  
912854-A420

## en Adapt Barrier Rings

**Contents:** Barrier rings  
**Product Description:** Barrier rings designed to improve the fit of an ostomy pouching system. Made with hydrocolloid skin adhesive.

**Indications and Intended Use:** This product is an accessory to an ostomy pouching system intended to help prevent drainage from getting under the skin barrier. Barrier rings help ostomy products conform to the skin.

**WARNINGS:** Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

**Notes:** The product may be cut, stretched, or stacked together to help improve fit of a pouching system. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see [www.hollister.com/authority](http://www.hollister.com/authority) or contact EC Rep or local distributor.


**Slim** Barrier ring with reduced thickness

**MD** Medical Device (as applicable).


Not made with natural rubber latex.  
CE mark as required

## INSTRUCTIONS FOR USE

### To prepare skin and product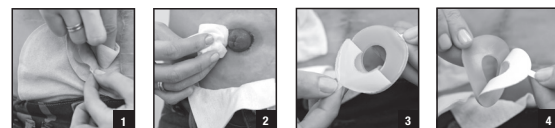

To apply

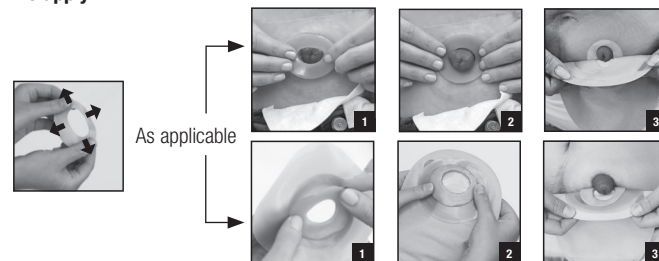To remove and dispose (Please follow all local disposal regulations)

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