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Ostomy barrier strips

**Product Description**

Barrier strips designed to improve the fit of an ostomy pouching system. Made with hydrocolloid skin adhesive.

**Indications and Intended Use**

This product is an accessory to an ostomy pouching system intended to help prevent drainage from getting under the skin barrier.

**WARNINGS**

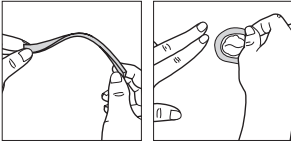
Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

**Notes**


The product may be cut to help improve fit of a pouching system. To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

**INSTRUCTIONS FOR USE**

**To apply:** Skin should be clean and dry. Remove protective covering. Roll, bend, stretch or cut, as required, before applying to peristomal area. Hold in place for several seconds.

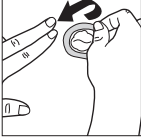**To remove and dispose:** (Please follow all local disposal regulations).

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©2020 Hollister Incorporated