## it LUBRIFICANTE PER STOMA

### Contenuto

Lubrificante per stoma

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Adapt™

REF 7740

912968-6-20  
©2020 Hollister Incorporated.  
Hollister

Hollister LLC  
Foxford Road, Ballina  
Co. Mayo, Ireland

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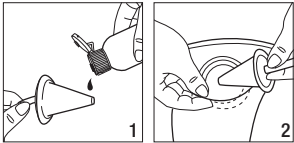
For reference only. Unauthorized use, disclosure or copying is prohibited.

## en STOMA LUBRICANT

### Contents

Stoma Lubricant

### Product Description

Stoma lubricant for use with a stoma cone irrigator. Ingredients: Water, Propylene Glycol, Hydroxypropyl Methylcellulose, Methyl Paraben and Sodium Hydroxide.

### Indications and Intended Use

This product is an accessory to an ostomy irrigation system intended to provide lubrication for insertion of stoma cone irrigator.

### Precautions

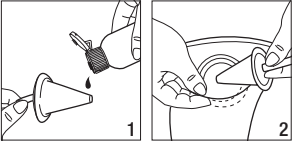
 Single patient - multiple use. Sharing the product with multiple users creates risk of infection.

### Notes

Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see [www.hollister.com/authority](http://www.hollister.com/authority) or contact EC Rep or local distributor.

### INSTRUCTIONS FOR USE

**To apply:** Coat end of irrigator cone with lubricant before inserting the cone into the stoma. If resistance is met, discontinue irrigation and consult your healthcare practitioner.

**To dispose:** Please follow all local disposal regulations. May use until bottle is empty.

 Medical Device (as applicable) CE mark as required

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