912913

Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Hollister ULC
Foxford Road, Ballina
Co. Mayo, Ireland

©2020 Hollister Incorporated.

en Skin Barrier

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es Lámina Protectora

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fr-ca Champ protecteur

it Barriera cutanea

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nl Huidplak

no Hudplate

sv Hudskyddsplatta

www.hollister.com/symbolglossary

912913-1120

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Inhalt

- HolliHesive Hautschutzplatte.
- Messschablone.

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en INSTRUCTIONS FOR USE

Contents

- HolliHesive skin barrier.
- Measuring guide.

Product Description: Regular/standard wear skin barrier designed to improve the fit of an ostomy pouching system. Designed to be used with an appropriate ostomy pouching system. Made with hydrocolloid skin adhesive.

Indications and Intended Use: This is an accessory to an ostomy pouching system intended to help ostomy products conform to the skin.

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

Notes: The product may be cut to help improve fit of a pouching system. To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier.

Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours.

Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation.


In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

MD Medical Device (as applicable)


Not made with natural rubber latex.
CE mark as required


To measure and cut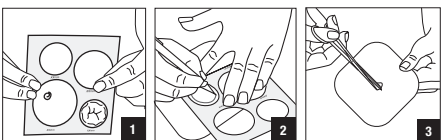


To prepare skin

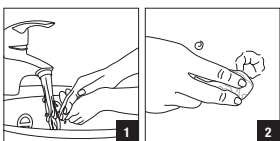To apply to skin

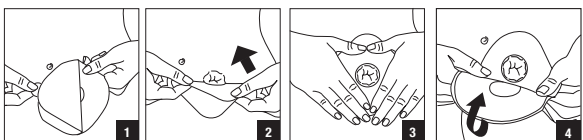To remove and dispose (Please follow all local disposal regulations)

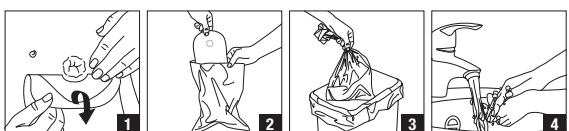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