Premium™

912911

Hollister Incorporated  
2000 Hollister Drive  
Libertyville, IL 60048 USA

Hollister ULC  
Foxford Road, Ballina  
Co. Mayo, Ireland

EC

REP

en Skin Barrier

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es Lámina Protectora

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fr-ca Champ protecteur

it Barriera cutanea

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www.hollister.com/symbolglossary

**Hollister**  
©2020 Hollister Incorporated.  
912911-1120

## en INSTRUCTIONS FOR USE

### Contents

- Premium skin barrier
- Measuring guide

**Product Description:** Regular/standard wear skin barrier designed to improve the fit of an ostomy pouching system. Designed to be used with an appropriate ostomy pouching system. Made with hydrocolloid skin adhesive.

**Indications and Intended Use:** This is an accessory to an ostomy pouching system intended to help ostomy products conform to the skin.

## WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

**Notes:** The product may be cut to help improve fit of a pouching system. To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier.

Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours.

Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation.

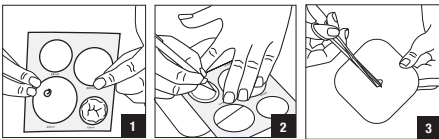
In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see [www.hollister.com/authority](http://www.hollister.com/authority) or contact EC Rep or local distributor.

**MD** Medical Device  
(as applicable)


Not made with natural rubber latex.  
CE mark as required

Number of units


### To measure and cut


### To prepare skin
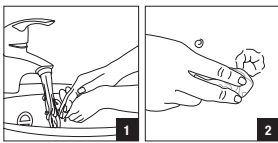
### To apply to skin
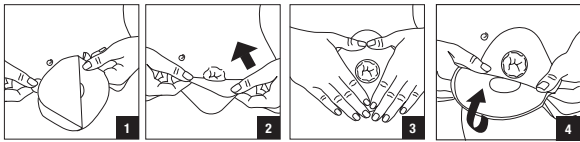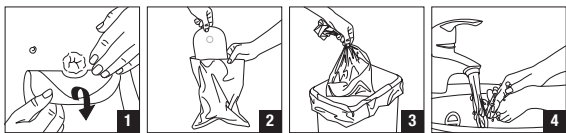
### To remove and dispose (Please follow all local disposal regulations)

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