(en) Peristomal Skin Cleanser

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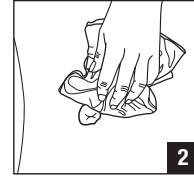
©2020 Hollister Incorporated.
913022-1220

(en) Peristomal Skin Cleanser

Contents: Peristomal Skin Cleanser**Product Description:** Skin cleanser for cleansing skin around a stoma.**INGREDIENTS:** WATER, POLYSORBATE 20, PROPYLENE GLYCOL, SODIUM LAUROAMPHOACETATE, SODIUM TRIDECETH SULFATE, CITRIC ACID, METHYLPARABEN, IMIDAZOLIDINYL UREA, DISODIUM EDTA, FD&C BLUE NO. 1, PROPYLPARABEN.**Indications and Intended Use:** This product is an accessory to an ostomy pouching system intended for skin management.**Precautions: For external use only.**

Use only as directed. For use on intact skin only.
Single patient - multiple use. Sharing the product with multiple users creates risk of infection.

Notes: Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

INSTRUCTIONS FOR USE:**To apply**

1. Spray directly to area to be cleaned.
2. Wipe peristomal skin area clean with cloth.
3. Be sure that the skin is thoroughly dry before applying new skin barrier.

To dispose. Please follow all local disposal regulations.

Not made with natural rubber latex.
CE mark as required

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