

## Respiflo™

Respiratory Exerciser

REF 8884717395

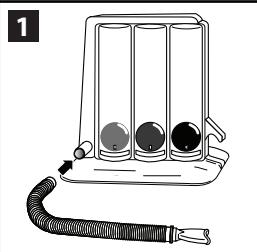en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
sv	Bruksanvisning
pt-PT	Instruções de utilização

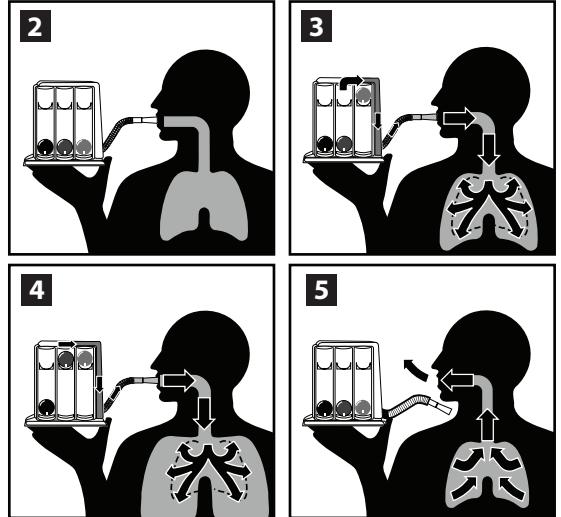**en**

**Description:** Your Respiflo™ respiratory exerciser has been scientifically constructed as a means of encouraging sustained maximal inspiration.

When you are experiencing pain following chest or abdominal surgery, for example, breathing tends to become shallow, and deep breaths are suppressed in an effort to minimize pain. In these instances it is important that you strive to resume your normal breathing pattern. Taking the deep breaths that you might ordinarily suppress may help prevent the possibility of respiratory complications.

Follow your physician's instructions and the directions with your Respiflo™ respiratory exerciser. If a problem is encountered related to achieving your normal breathing pattern, contact your physician.

Dispose of the device in accordance with applicable national regulations.

**Indications/Intended Use/Purpose:** The Respiflo™ respiratory exerciser is used by patients following thoracic surgery or in other conditions to assist the lungs in returning to normal functioning. The device contains graduations; however these are used for self-evaluation/motivation of the patient and are not for diagnostic use. The respiratory exerciser is for single patient - multiple use. Duration of use should not exceed twenty-nine (29) days. Covidien has not substantiated use of this device beyond 29 days. Decisions about respiratory exercisers should be made by the responsible physician or designee using accepted medical techniques and judgment.

**Contraindications:** None known.

**How to assemble your Respiflo™ respiratory exerciser:**

1. Remove Respiflo™ components from package.
2. Attach the free end of the tubing (without mouthpiece) to the stem at the base of unit (1).

NOTE: Before and after each use, rinse only the mouthpiece with warm water and shake it dry. Between uses, place the unit in its bag and close top.

**Instructions**

1. With the unit in an upright position, **Exhale** normally and then place your lips tightly around the mouthpiece (2).
2. **FOR A LOW FLOW RATE** — Inhale at a sufficient rate to raise only the ball in the first chamber while the second chamber ball remains in the rest position (3). Hold inhale for three seconds.
3. **FOR A HIGHER FLOW RATE** — Inhale at a rate sufficient to raise the first and second chamber balls while the third chamber ball remains in the rest position (4). Hold inhale for three seconds or as long as comfortably possible.
4. **EXHALE** — Remove mouthpiece from lips and exhale normally (5).

Following each prolonged deep breath take a moment to rest and breathe normally. Then repeat the exercise as directed by your physician.

NOTE: Patients who find it difficult to raise the ball or balls while holding the unit in an upright position may tilt the unit forward slightly for ease of use.

**Adverse Events:** The following adverse events have been reported to be associated with the use of respiratory exercisers, like Respiflo™, listed in order of frequency and severity: Pain, tissue trauma, foreign body, airway obstruction, respiratory distress, allergic reaction/anaphylaxis, pneumothorax, infection, and/or syncope/dizziness. Any serious incident that occurs in relation to the device should be reported to Covidien and to the national competent authority.

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Απορρίψτε τη σύσκευη ανάπονης με τους ισχύοντες εθνικούς Covidien AG.™ brands are trademarks of their respective owners. Other brands are trademarks of a Covidien company.